

1-Qt., 1.5-Qt., 1, 2 & 3-Gallon Sharps Mailback System

SYSTEM COMPONENTS (Save for packaging and return mailing.)

- Prepaid return shipping box with pre-attached USPS mailing label (return postage is prepaid)
- Inner box and 4 mil plastic bag liner
- Red sharps collection containers with absorbent pad inside
- Tape, twist-tie
- Tracking document on outside of shipping box
- Instructions



INSTRUCTIONS (Read instructions including all warnings completely before using this system.)

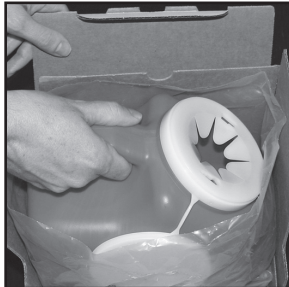


Fig. 1: Remove container from packaging and place at point of use. DO NOT REMOVE ABSORBENT PAD.

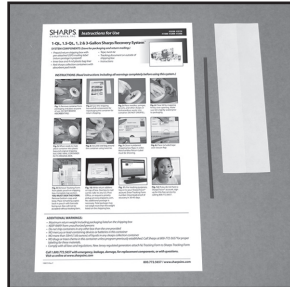


Fig. 2: Save the shipping box and all components for repackaging the container for return shipping.

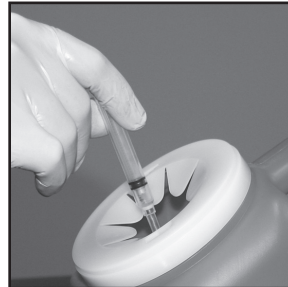


Fig. 3: Place needles, syringes, lancets, and other sharps or biohazardous waste into container. DO NOT OVERFILL.

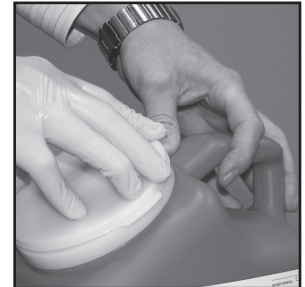


Fig. 4: Close lid by snapping firmly over opening. Make sure lid is tightly sealed prior to packaging.



Fig. 5: When ready to mail, place container into plastic bag and original shipping box. ONLY MAIL CONTAINER IN ITS ORIGINAL BOX.

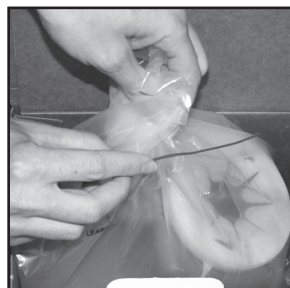


Fig. 6: Securely seal bag around the container using twist-tie.

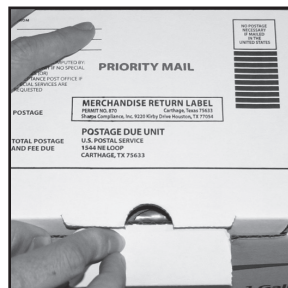


Fig. 7: Close numbered shipping box flaps in order. Merchandise Return Label must be showing.



Fig. 8: Place included tape over closure flap.



Fig. 9: Remove Tracking Form from plastic pouch on shipping box. Complete top section. **YOU MUST SIGN THE FORM.** Remove bottom copy and keep. Place remaining copies back in pouch with barcode facing out. Box will not be accepted without tracking form.

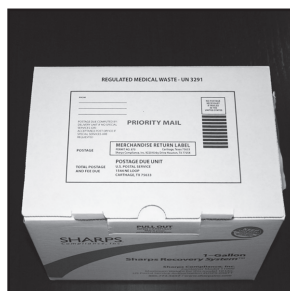


Fig. 10: Give box to mail carrier, take to any U.S. Post Office, or request a priority pickup at www.sharpsmws.com. No additional postage is necessary. Total package may not weigh more than the weight listed on the shipping box.



Fig. 11: For tracking purposes, log on to your SharpsTracer® account. Enter Tracking Form number. Download proof of recovery in 30-45 days.

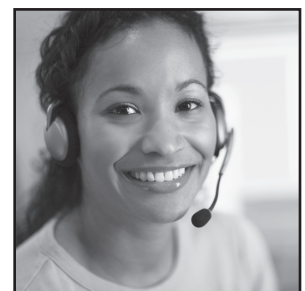


Fig. 12: If you do not have a SharpsTracer® account, sign up for this free service by calling 800.772.5657.

ADDITIONAL WARNINGS:

- Maximum return weight including packaging listed on the shipping box
 - Customers returning systems with weights in excess of the maximum specified weight are subject to an overweight surcharge
 - KEEP AWAY from unauthorized persons
 - Do not ship containers in any other box than the one provided
 - NO mercury or lead-containing devices or batteries in this container
 - NO more than 50ml (1.66 ounces) of liquids in any sharps collection container
 - NO drugs or trace chemo in this container unless program previously established. Call Sharps at 800-772-5657 for proper labeling for these materials
 - Comply with all laws and regulations. New Jersey regulated generators attach NJ Tracking Form to Sharps Tracking Form
- POST OFFICE WILL NOT ACCEPT YOUR BOX WITHOUT THE MANIFEST TRACKING DOCUMENT.**

Call 1.800.772.5657 with emergency, leakage, damage, for replacement components, or with questions.
Visit us online at www.sharpsmws.com